

NIHON KOHDEN


MEK-3DL HEMATOLOGY CONTROL 2.0mL

ASSAY SHEET

LOT	B252L
	2025-05-05
OPEN	14DAYS

MEK-1301,1302,1305 Barcode Entry

The barcodes may be read in any order.
If the barcode is difficult to scan, place this assay sheet on a flat surface, keep the barcode reader and barcode vertical, and scan within a distance of 2cm.

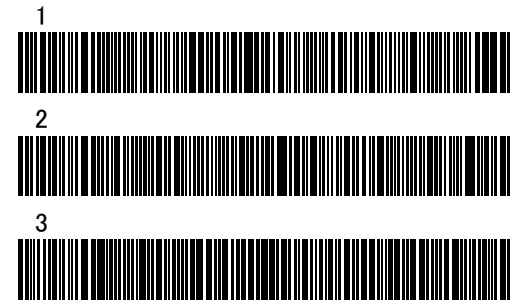

		RANGE	MEK -6400 -6410 -6420 -6450	MEK -6500 -6510 -6550	MEK -1301 -1302 -1305
WBC	10 ³ /μL	±0.6	2.1	2.1	2.08
RBC	10 ⁶ /μL	±0.15	2.47	2.47	2.43
HGB	g/dL	±0.4	6.0	6.2	6.08
HCT	%	±2.0	18.5	18.5	20.6
MCV	fL	±9.0	74.7	74.7	84.9
MCH	pg	±2.4	24.3	25.1	25.0
MCHC	g/dL	±3.0	32.4	33.5	29.5
PLT	10 ³ /μL	±30	66	66	75.6
MPV	fL	±3.0	5.6	8.2	8.2
RDW-CV *	%	±3.0	17.5	17.5	16.0
LY%	%	±10.0	60.8	60.8	59.75
MO%	%	±5.0	7.5	7.5	5.16
GR%	%	±10.0	31.7	31.7	35.09
LY	10 ³ /μL	±0.5	1.3	1.3	1.24
MO	10 ³ /μL	±0.2	0.2	0.2	0.11
GR	10 ³ /μL	±0.5	0.6	0.6	0.73

MEK-1305
QC ESR
43±17 mm

Manual Registration	
Lot	252
Level	Low
Control	MEK-3D

* RDW-CV is displayed as RDW on MEK-6400,6410,6420,6450.



<https://www.nihonkohden.com/spi/index.html>



NIHON KOHDEN CORPORATION

1-31-4 NISHIOCHIAI, SHINJUKU-KU
TOKYO 161-8560 JAPAN

